



Instructions For Use CEM-1-IFU

Rev. Date: Feb. 29, 2016

Revision: 4

Page 1 of 2


P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - www.scytek.com

Eosinophil-Mast Cell Stain Kit

Description: *The Eosinophil-Mast Cell Stain Kit is intended for simultaneous visualization of eosinophils and mast cells.*

Mast Cells: *Bright Blue*
Eosinophils: *Red*
Nuclei: *Blue*

Uses/Limitations: *Not to be taken internally.
For In-Vitro Diagnostic use only.
Histological applications.
Do not use past expiration date.
Use caution when handling reagents.
Non-Sterile.*

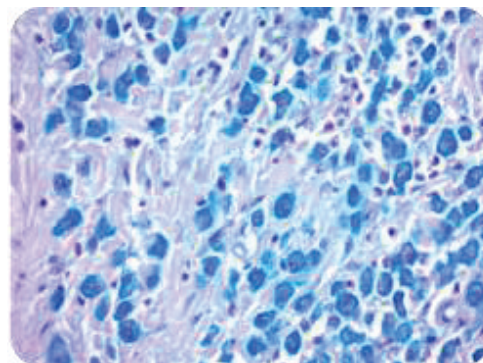

Control Tissue: *Any well fixed section cut 3-5 microns or any air dried smear on clean slide.*

Availability/Contents:

<u>Item #</u>	<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
AST125	Astra Blue Solution	125 ml	18-25°C
VRN125	Vital New Red Solution	125 ml	18-25°C
HMM125	Hematoxylin, Mayer's (Lillie's Mod.)	125 ml	18-25°C

Precautions: *Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.
Use in chemical fume hood whenever possible.*

Procedure (Standard):

1. *Deparaffinize sections if necessary and hydrate to distilled water.*
2. *Stain tissue section with Astra Blue Solution for 1-3 minutes.*
3. *Rinse quickly in 2 changes of distilled water.*
4. *Stain tissue section with Vital New Red Solution for 30 minutes with frequent agitation.*
5. *Rinse quickly in 2 changes of distilled water.*
6. *Counterstain in Hematoxylin, Mayer's (Lillie's Modification) for 15-30 seconds.*
7. *Rinse slide in running tap water for 2 minutes followed by one dip in distilled water.*
8. *Dehydrate quickly through 3 changes of absolute alcohol.*
9. *Clear, and mount in synthetic resin.*

Storage: 18° C



25° C




ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.




 EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands

References:

1. Kermanizadeh, P., et al. *A differential staining method for mast cells and eosinophils in murine intestine, liver and spleen. Parasitology Today, 1995, May; 11(5): pages 194-196.*
2. Ball, M.T., Hay, J.: *Simultaneous Demonstration of Eosinophils, Granulocytes, and Mast Cell in Tissue Sections. Annals of Tropical Medicine and Parasitology., 84: pages 195-196, 1990.*
3. Strobel, S., et al. *Human intestinal mucosal mast cells: evaluation of fixation and staining techniques. Journal of Clinical Pathology, 1981; 34: pages 851-858.*

Storage: 18° C  25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

CE 

 EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands