

ScyTek
Laboratories

Instructions For Use HWI-IFU

Rev. Date: Nov. 17, 2009

Revision: 3

Page 1 of 2

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - www.scytek.com

Hematoxylin, Weigert's Iron Kit

Description: Weigert's Iron Hematoxylin is intended to be used with various special stain kits and procedures. This product is supplied as a two component (equal volumes of parts A and B) system that is mixed prior to use and results in blue/black nuclei.

Results: Nuclei: Blue/Black

Uses/Limitations: For In-Vitro Diagnostic use only.
Histologic/Cytologic applications.
Do not use past expiration date.
Use caution when handling these reagents.

Control Tissue: Any well-fixed tissue section.
Cell smear.

Storage: Room Temperature

Availability/Contents:

<u>Item #</u>	<u>Volume of Parts A & B</u>	<u>Combined Volume</u>
HWI-1	125 ml each	250 ml
HWI-2	500 ml each	1000 ml
HWI-3	1000 ml each	2000 ml

Suggested But Not Included:

BRT500 Bluing Reagent

Precautions: Flammable.
Keep away from open flame.
Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.
Use in chemical fume hood whenever possible.

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.



 EmergoEurope (31)(0) 70 345-8570
Molsstraat 15
2513 BH Hague, The Netherlands

Instructions For Use
HWI-IFU

Rev. Date: Nov. 17, 2009

Revision: 3

Page 2 of 2

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - www.scytek.com

Procedure:

1. Immediately before use, mix equal volumes of Part A and Part B. (Note: Do not reuse mixed stain)
2. Using graded alcohols, rehydrate tissue section to distilled water.
3. Stain slide in combined Hematoxylin, Weigert's Iron for 2-5 minutes.
4. Rinse for 2 minutes in several changes of tap water.
5. Dip slide in Bluing Reagent (BRT) for 30 seconds.
6. Rinse slide in distilled water.
7. Dehydrate in absolute alcohol.
8. Clear, and mount in synthetic resin.

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.



EC REP EmergoEurope (31)(0) 70 345-8570
Molsstraat 15
2513 BH Hague, The Netherlands