



Instructions For Use TGG-IFU		
Rev. Date: Aug. 31, 2016	Revision: 4	Page 1 of 3

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

Trichrome Stain Solution (Green) (Modified Gomori's)

Description: The Trichrome Stain Solution (Green) is a component of the Trichrome Stain Kit (Modified Gomori's) and is intended for use in the histological visualization of collagenous connective tissue fibers in tissue sections.

Collagen:	Green
Muscle Fibers:	Red
Nuclei:	Black/Blue (Hematoxylin)

Uses/Limitations: For In-Vitro Diagnostic use only.
Histological applications.
Do not use past expiration date.
Use caution when handling this reagent.

Control Tissue: Lung
Uterus
Small Intestine
Appendix

Storage: Room Temperature (18-25° C.)

Availability/Contents:

<u>Item #</u>	<u>Volume</u>
TGG125	125 ml
TGG500	500 ml
TGG999	1000 ml

Required but Not Included:

<u>Item #</u>	<u>Kit Contents</u>
BNF125	Bouin's Fluid
HWI-250	Weigert's Iron, Hematoxylin
AAD250	Acetic Acid Solution
AAL250	Acid Alcohol Solution

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.



Authorized Representative in

Europe



(Regulatory affairs only)

EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands

Instructions For Use TGG-IFU

Rev. Date: Aug. 31, 2016

Revision: 4

Page 2 of 3

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

Precautions: Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.
Use in chemical fume hood whenever possible.

Procedure (Standard):

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Preheat Bouin's Fluid (BNF) in a water bath to 62° - 64° centigrade in a fume hood or very well ventilated area.
3. Place slides in preheated Bouin's Fluid (BNF) for 60 minutes followed by a 10 minute cooling period.
4. Rinse slide in tap water until section is completely clear.
5. Rinse once in distilled water.
6. Stain slide with working Weigert's Iron Hematoxylin (HWI) for 5 minutes.
7. Rinse slide in tap water for 2 minutes.
8. Apply Acid Alcohol Solution (AAL) to slide for 5 seconds to adjust pH.
9. Rinse slide in tap water for 1 minute.
10. Rinse slide in distilled water.
11. Apply Trichrome Stain Solution (Green) to tissue section for 15 minutes.
12. Rinse slide in distilled water.
13. Apply Acetic Acid Solution (AAD) to tissue section for 10 seconds.
14. Rinse slide in distilled water.
15. Dehydrate through graded alcohol, clear, and mount in synthetic resin.

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.



Authorized Representative in

Europe



(Regulatory affairs only)
EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands



Instructions For Use TGG-IFU

Rev. Date: Aug. 31, 2016

Revision: 4

Page 3 of 3

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

References:

1. Gomori, G.L., A rapid one-step trichrome. Amer J of Clinical Pathology. Vol. 20: page 661, 1950.
2. Manual of Histologic Staining Methods of AFIP. 3rd Edition, McGraw-Hill Publishing. Page 93, 1968.
3. Elbadawi, A., Hexachrome modification of Movat's stain. Stain Technology, Vol. 51: pages 249-253, 1976.
4. Carson, F.L., Histotechnology; A Self-Instructional Text, 2nd Edition. ASCP Press, Chicago, IL. Pages 117-121, 1996.

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.



Authorized Representative in

Europe



(Regulatory affairs only)
EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands