
CytoMatrix

CytoMatrix is a porous material with the property of capturing and storing cytological preparations such as material from fine needle aspirates or cell pellets from body fluids inside its three-dimensional structure. The high effectiveness of CytoMatrix is based on the high affinity of the synthetic scaffold, which effectively retains cytological samples in the matrix and keeps them dispensed on the inner surface, as well as on the wide pores which allow extensive diffusion of the fixative, whereby the morphology of cytological samples is particularly well preserved. Sample material from fine needle aspirates is transferred to the CytoMatrix immediately after sampling, fixed in formalin and embedded in paraffin. Since the sample does not have to be processed further, cell material from fine needle aspiration remains unaffected by any operator and is retained in the CytoMatrix to its full extent without manipulation. Paraffin sections of cytological samples included and preserved in CytoMatrix can be examined in various routine diagnostic tests such as hematoxylin-eosin (H&E) staining, immunohistochemistry (IHC), FISH or in molecular biology. CytoMatrix is particularly helpful for optimal management of scarce cytological samples in order to ensure a complete and secured diagnosis.

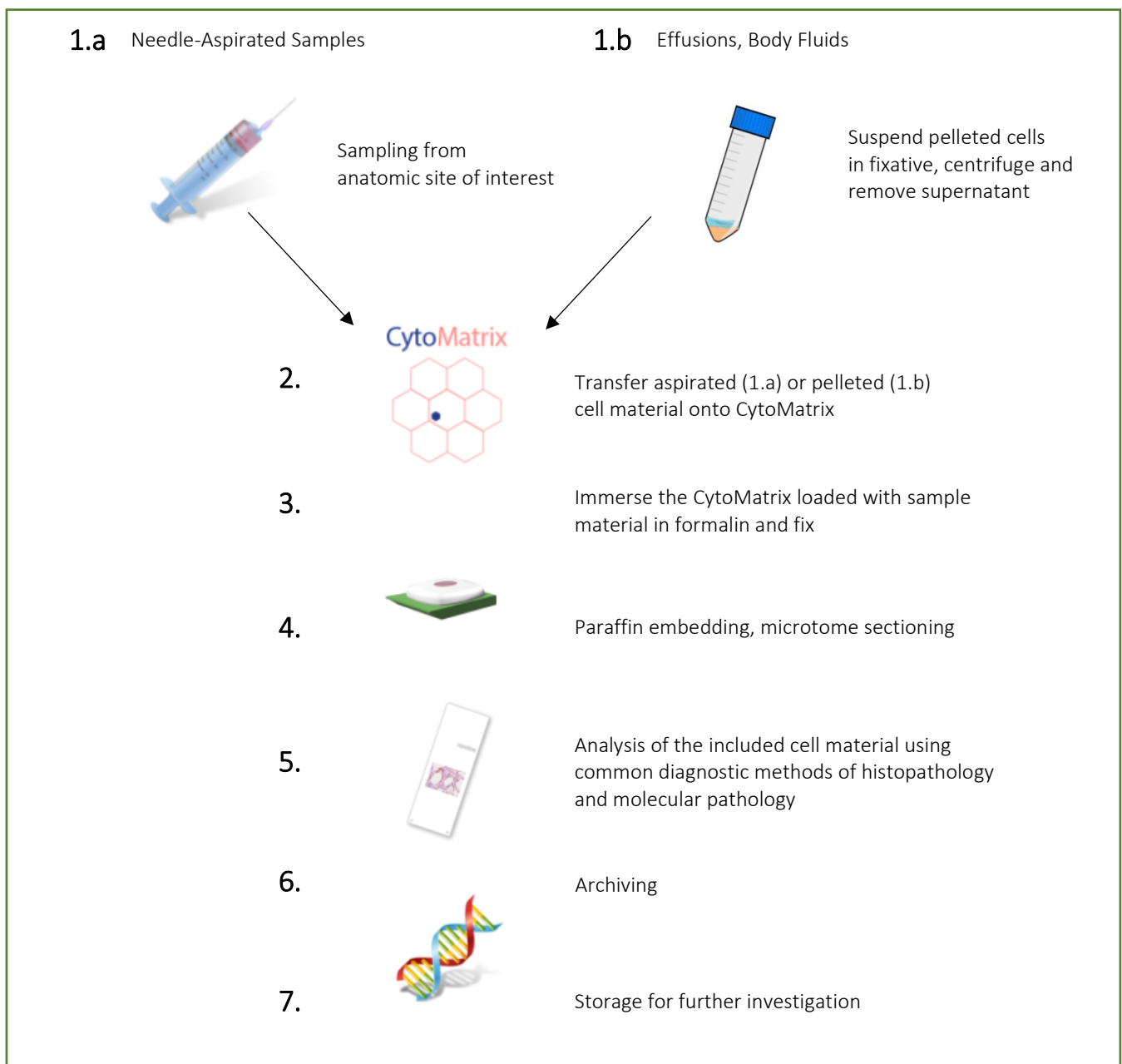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dianova GmbH